



Rugby PPE Response Initiative

VISOR CARE ADVICE

23rd April 2020

Dear Colleague

The Visor we offer is on a “something is better than nothing” basis.

Initial user feedback has been very positive. Our aim is to provide all front-line Health and Social Care Colleagues in Rugby with a PPE visor as soon as we can, and until full NHS supplies are fully available.

The visors can be washed in hot soapy water (60°C), air dried and then sprayed with alcohol or cleaned with alcohol wipe.

The elastic at the back **MUST NOT** be recycled. Wear fresh gloves to doing this work and a face mask.

Follow your organisational guidance on putting on and off PPE.

Safely dispose of non-recyclable items

Lastly, can you please send us feedback as we are using a Continuous Quality Improvement approach.

Please look after yourself.

Best Wishes

Dr Keith Edgar FRCGP Project Lead

Disclaimer:- Whilst precautions have been taken in the design and manufacture of this equipment, Rugby PPE Response disclaims liability for the functioning or capability of the equipment or for any errors or omissions in the design of the equipment. Rugby PPE Response does not accept liability, whether in contract, tort (including negligence), breach of statutory duty, or otherwise, for any loss or damage whatsoever incurred by the user of the equipment or by Rugby PPE Response in connection with the use of this equipment. In return for the supply of the equipment recipient agrees to indemnify Rugby PPE Response and hold it harmless in respect of any claim, loss or expense incurred by Rugby PPE Response arising out of the use of this equipment by recipient organisation or its staff.